



Neuro IPM Release Notes

Release Date: 08/29/2023

Version: 2.7.4

Included in the release:

New Development:

- 1. Addition of a progress bar
- 2. Updated step names
- 3. Addition of DataDog for frontend monitoring


Issues/Bug Fixes:

4. Tablet bug resolved related to available camera streams.

Details:

1. Addition of a progress bar

A progress bar and step counter has been added to the user interface. Participants can now determine what step number they're on with a visual indication of progress.

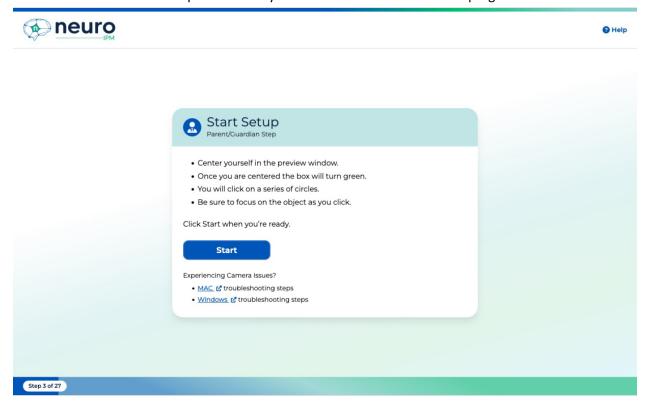





2. Updated Step names

The step names have been updated to provide a better indication of the steps' purpose. The updated step names are as follows:

- Step 1: Welcome (Introduction video)
- Step 2: Setup Requirements (Hardware/Environmental Requirements)
- Step 3: **Start Setup** (click the circle objects)
- Step 4: Finish Setup (smooth yellow dot video)
- Step 5: **Sample Video** (practice performance measure)
- Step 6: Preparing your child (Your Child's Experience-informative video)
- Step 7: Your Child's Turn (video preview/headbox)
- Step 8: **Child Setup** (child's video with moving objects)
- Step 9-Step 27: Video Tasks (includes SA, RV, PS, SWR, & VWM)

3. Datadog implemented

Datadog has been added for frontend monitoring. DataDog provides a user's device information, browser type, and allows for the playback of the user's screen capture. This all allows for better troubleshooting and error investigation.

4. Bug Fix:

An Android tablet bug was resolved related to the devices available camera streams.