

Neuro IPM Release Notes

Release Date: 06/18/2024

Version: 3.8.1

Included in the release:

New Development:

1. Warning and stop for users when a device's width is less than 960 pixels
2. Update to Help tab

Bug Fixes:

3. Resolved issue with the participant ID field not highlighting on the authorize screen.

New Development

1. Warning and stop for users when a device's width is less than 960 pixels.

Details:

On launch of the application, if a device with a width less than 960 pixels is detected, the participant is provided a warning and not allowed to proceed. The new feature is intended to stop cell phones from being used to complete the webcam tasks.

10in Screen Size Required

The application requires your device to have a screen that is at least 10 inches. Please switch to a tablet, laptop, or desktop (with camera) to proceed.

If the warning is received, the participant will need to retry the webcam tasks using a device with a larger screen. If a larger device is not available, the participants should contact their clinical provider.

2. Update to Help tab

Details:

The “Parent / Caregiver Instructions for Use” manual has been added to the help button.

Bug Fixes:

3. Resolved issue with the participant ID field not highlighting on the authorize screen.

Details:

An issue has been resolved where the participant ID field could not be highlighted on iPadOS devices.